



FreeStyle

Lite

Blood Glucose Monitoring System

1 Wash hands

2 Hold lancing device

3 Press lancing device against skin

4 Push lancing device back

5 Rotate lancing device 3x

6 Insert test strip

7 Push lancing device forward

8 Press lancing device against skin

9 Push lancing device back

10 Open lancing device container

11 Insert lancing device into meter

12 Meter displays result

13 Push lancing device forward

14 Push lancing device against skin at $> 45^\circ$

15 Meter displays result with alarm icon

16 Meter displays result with alarm icon

17 Dispose of lancing device in a sharps container

Abbott Diabetes Care

Klantenservice
0800 - 022 88 28 (gratis)

 **Abbott**
Diabetes Care